- Ⓢ Operating manual for the Pressalit Care Indivo kitchen
- Ⓢ Brugsanvisning for Pressalit Care Indivo-køkkenet
- Ⓢ Bruksanvisning for Pressalit Care Indivo kjøkken
- Ⓢ Bruksanvisning för kök Pressalit Care Indivo

keep living

UK

Before initial operation

Congratulations on your purchase of new lifts for your kitchen.

- Before initial operation, please affix the safety labels found in this booklet.
- This booklet should be kept in immediate vicinity of the lifts.
- The user, helper and therapist should read and understand this operating manual before using the lifts for the first time.
- The lifts are designed for indoor use only.
- For further information, please refer to www.pressaliticare.com

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UK**Application**

Height adjustment	4
Safety plates	6
Safety bars	7
Max. load	8

Maintenance

Weekly maintenance of lifts	9
Cleaning	10
Advice on safety	11

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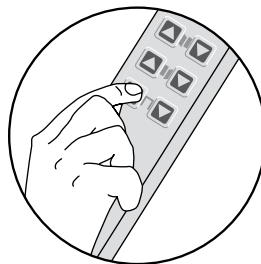
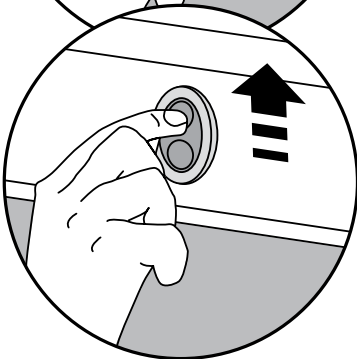
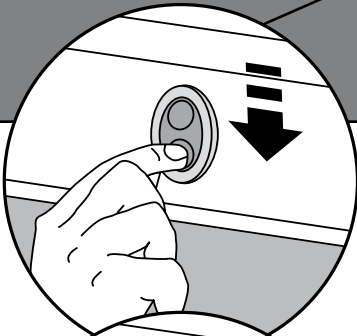
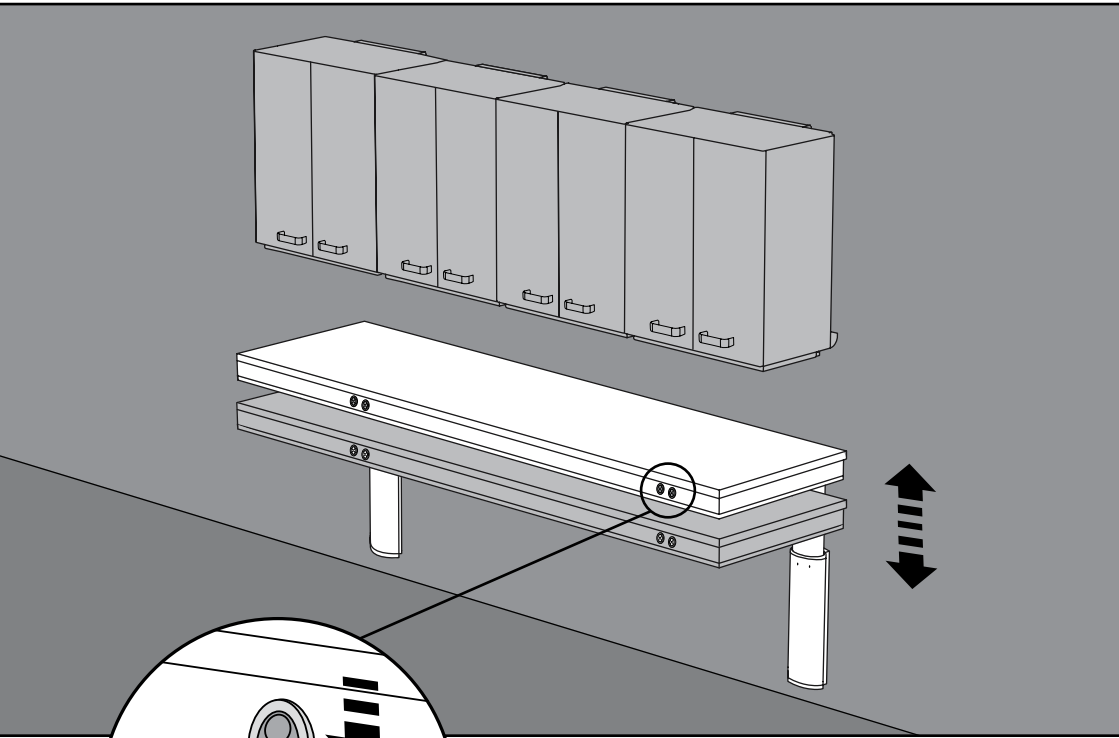
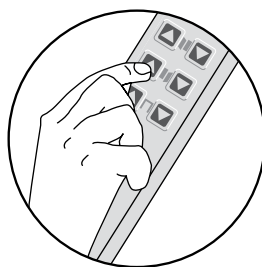
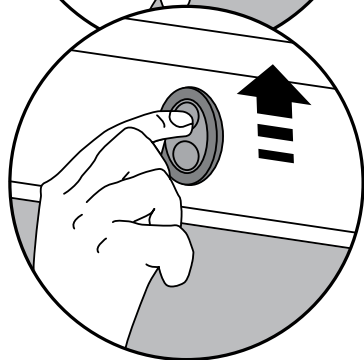
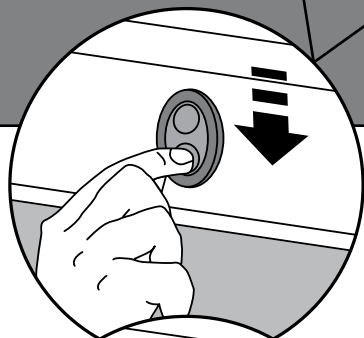
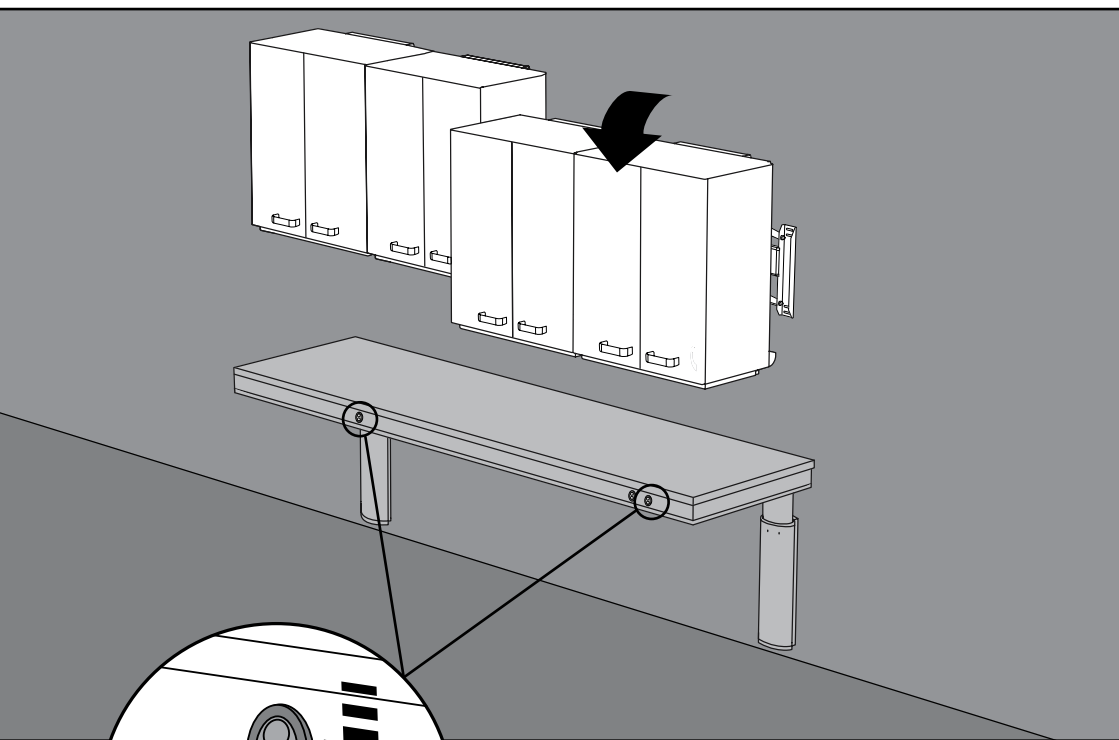
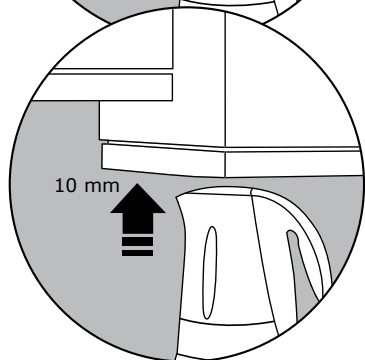
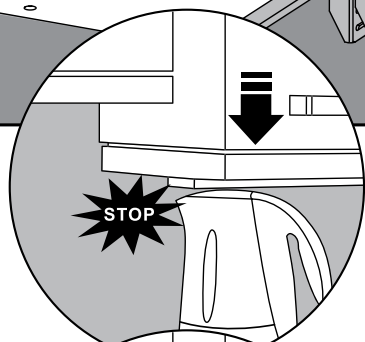
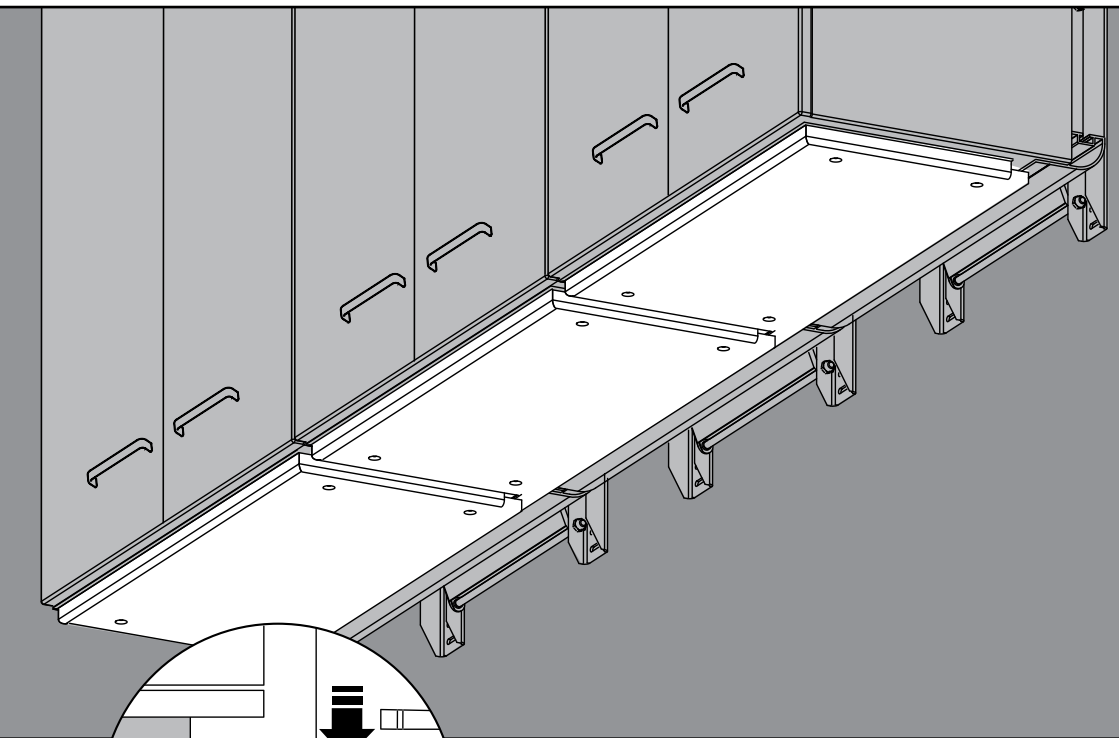
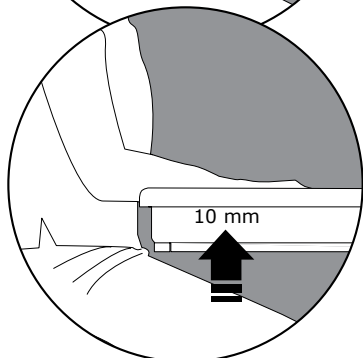
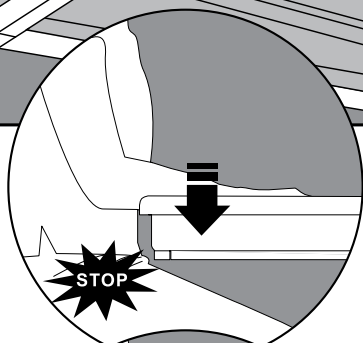
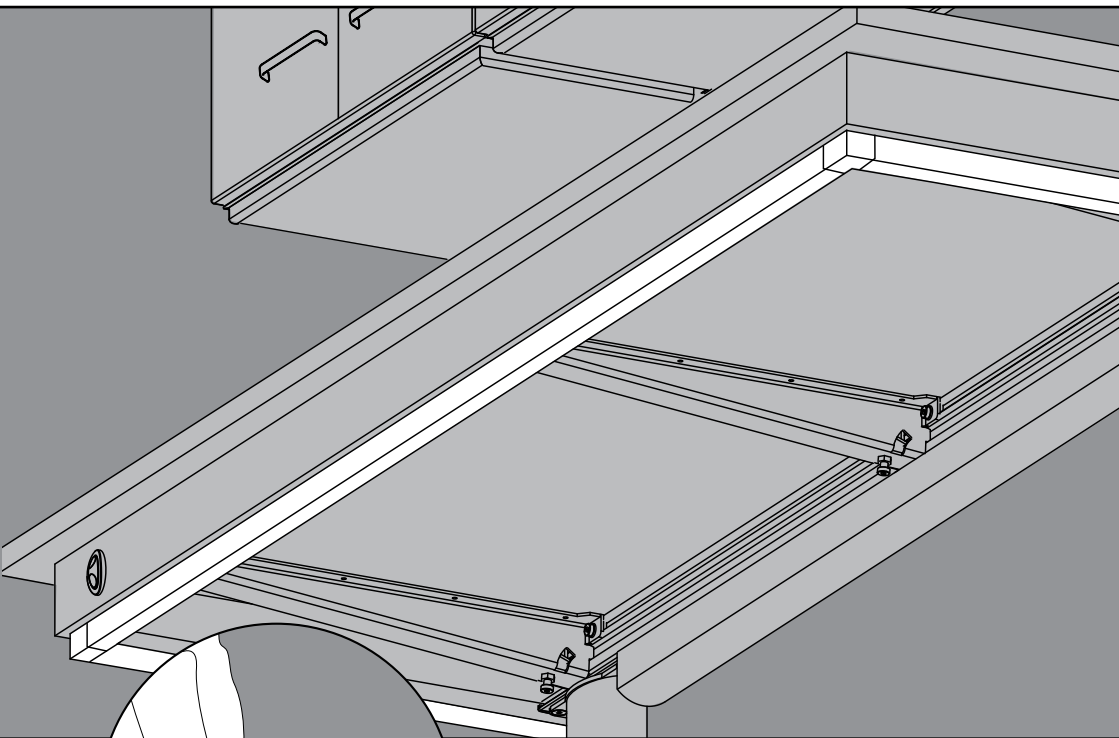
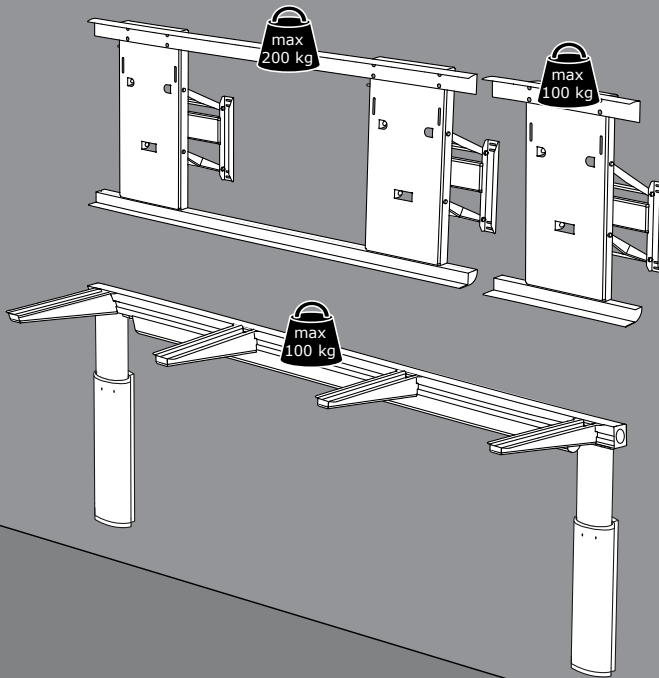
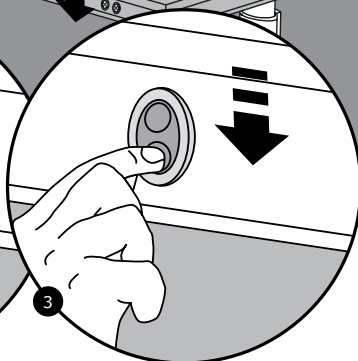
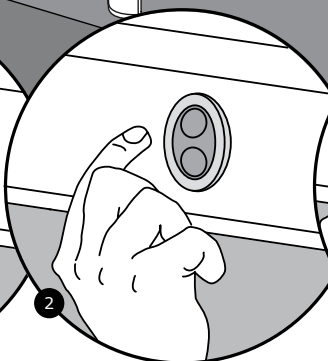
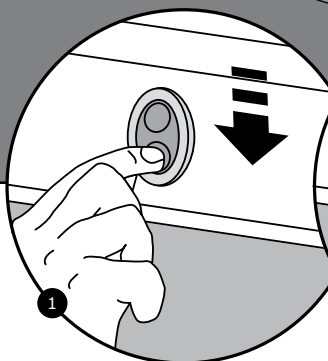
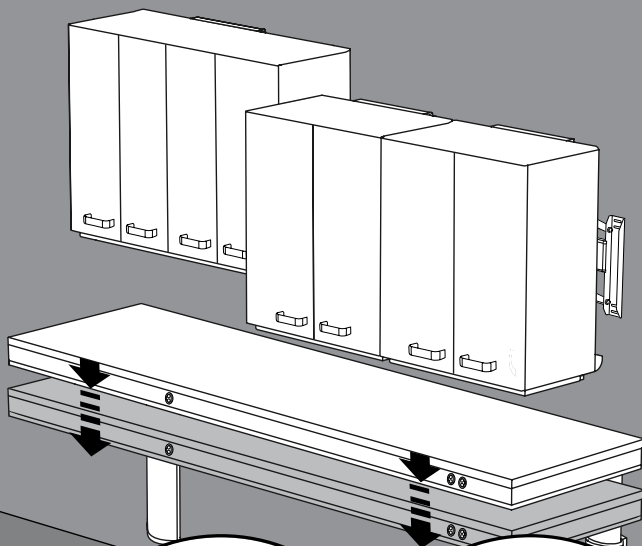
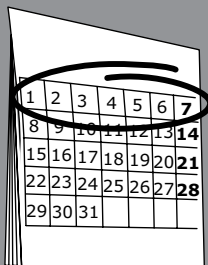
UK Hold in the switch until the table top stops in the lowest position.

Release the switch.

Press and hold in the switch once more for 5 seconds. Perform the same operation for the wall cupboards.

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
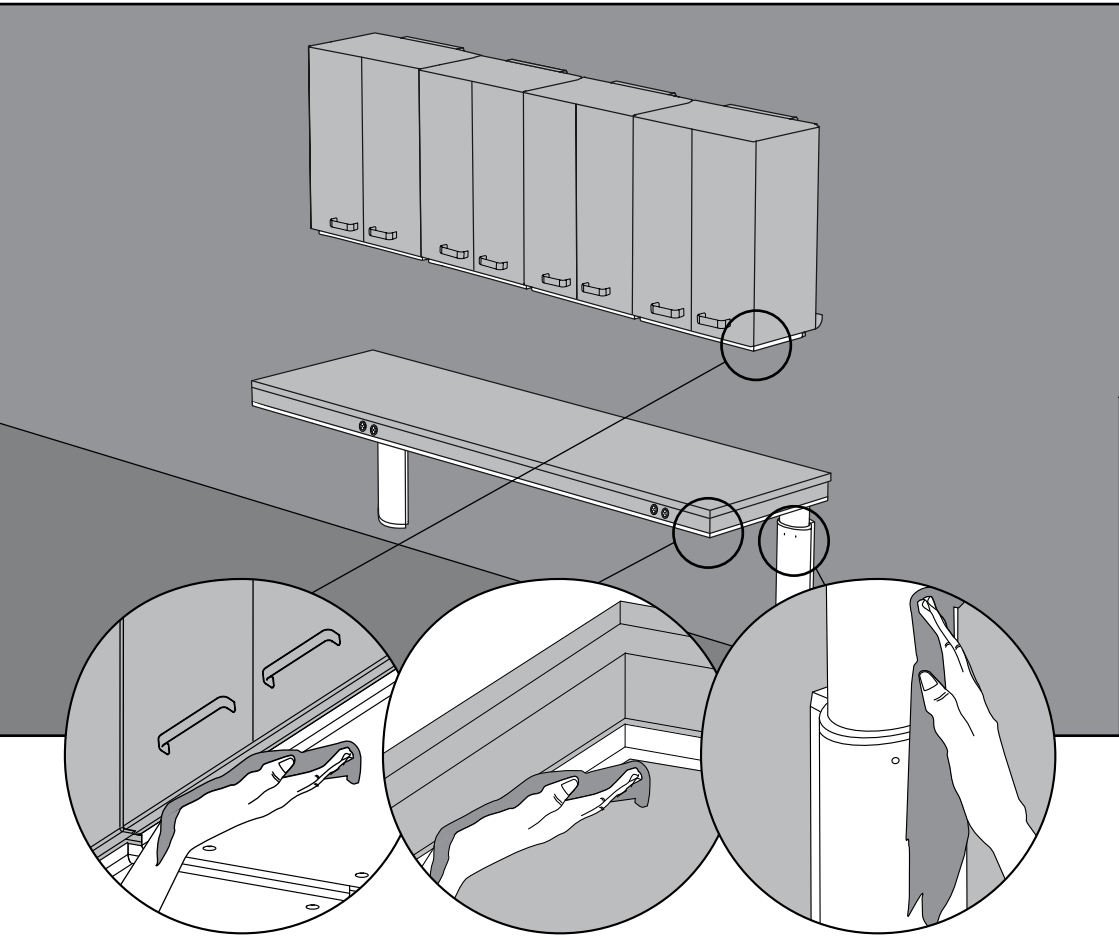
UK Use ordinary soapy water only (avoid abrasive, etching, chloride cleaning agents).

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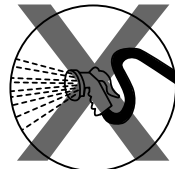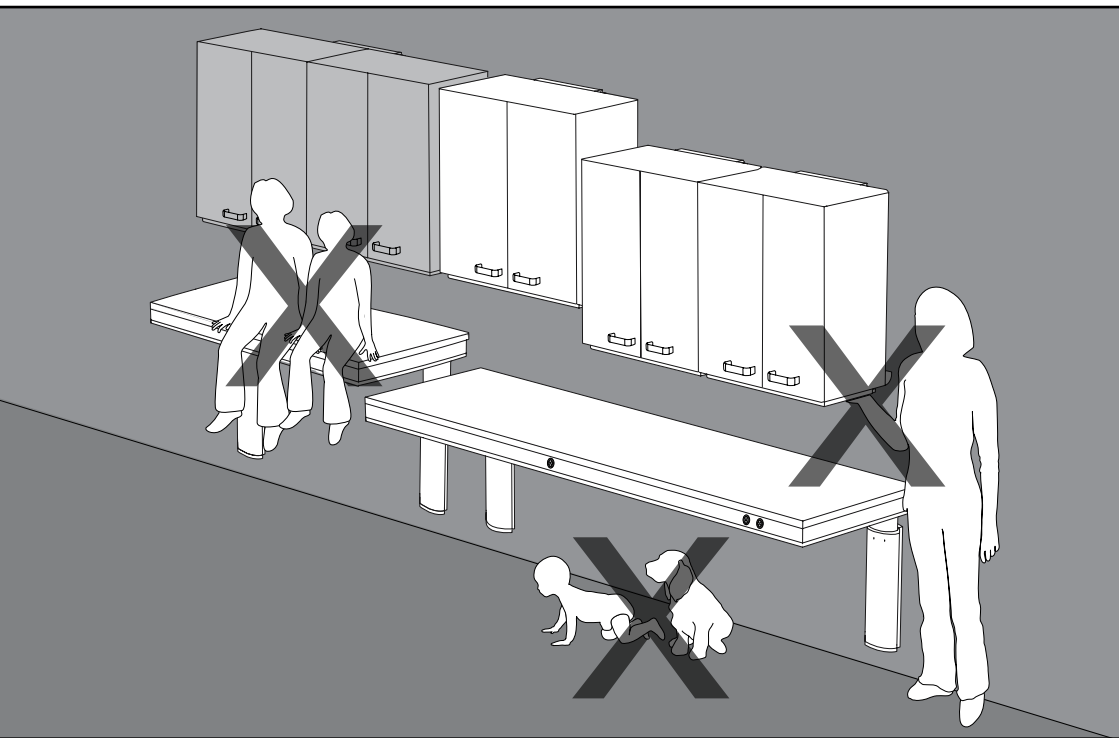
UK Do NOT use the safety plates and safety bars under the wall cupboards and the table top as a stop function!

During height adjustment, objects must not rest on two surfaces at the same time.

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UK Troubleshooting

Problem	Solution
<p>The height of the table top or wall cupboards cannot be adjusted</p>	<p>1. Check that the safety bar under the table top or the safety plates under the wall cupboards have not been activated. If applicable, try to activate the safety bars or safety plates. It should be possible for them to move about 5 mm. If a sink and installation cover have been fitted, check that the installation cover does not collide with any bevelled cover moulding. If the problem persists, go to item 2.</p> <p>2. Check that all operation switches can be operated without obstruction and that they return to a neutral position. Movement may be hampered by dirt or grease. If the problem persists, go to item 3.</p> <p>3. Reset the lifts (see illustration p. 9). a) Activate the switch until the table top is in its lowest position. Release the switch. b) Activate the switch again for 5 seconds. The lifts for the table have now been reset.</p> <p>If height-adjustable cupboards have been installed, do the same with the cupboards one at a time. Please note that the wall cupboards may be lowered very slowly until resetting has taken place and that it may not be possible in the sequence mentioned. The cupboards may have to be reset first.</p> <p>It may be necessary to reset the lifts several times. If the problem persists, go to item 4.</p> <p>4. Check that all control boxes are provided with power, and that all plugs have been connected. The control box will typically be located behind the wall cupboards and under the table tops. If the problem persists, go to item 5.</p> <p>5. Disconnect the power if possible. Reconnect power after 5 seconds. After power disconnection, go to item 3.</p>
<p>Cupboards or table top stop and will not go up any further</p>	<p>Check that the cupboards or table top are not overloaded - see page 8. If this is the case, remove some weight. Subsequently, the table top or wall cupboards should be reset – see item 3.</p>
<p>Safety bar or safety plate does not function</p>	<p>Contact supplier.</p>
<p>Light under wall cupboards does not function</p>	<p>Remove the cover with a slight tug, and replace any defective tubes. If this does not solve the problem, contact the supplier.</p>

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E-mail mail@pressalitcare.com

A division of Pressalit Group A/S

For other addresses, please visit www.pressalitcare.com

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Pressalit Care A/S